1. Product Name

Musical Positioning Cushion by ROMPA®

2. Product Code

15781, 17781

3. Colour

Available in a wide choice of fabrics and colours

4. Brief Description

A quality addition to the ROMPA® product range which gives tactile and stimulation whilst providing comfortable support

5. Contents

- 1 x cushion
 - 1 x 2-way speaker control
- A music system is required to operate this product (to be purchased separately)

6. Snoezelen® Stimulations

- Touch

7. Best Use

Use indoors with a music system and an amplifier.

8. Compatible ROMPA® Products

21986 - CD Music System

14971 - Amplifier

9. Starting Up

- Place the cushion on a suitable floor.
- Insert the two leads from the integral speakers into SPEAKER # 1 OUT on the 2-way switch.
- Insert leads from the external speakers into SPEAKER # 2 OUT on the 2-way switch.
- Insert leads from music system into SPEAKER INPUT on the 2-way switch.
- Plug in your music system.
- Select the required operation (CD/radio etc.) and adjust volume to required level.

10. Detailed Description

A quality addition to the ROMPA® product range which gives tactile stimulation whilst providing comfortable support. The bean bag envelopes the user so the vibrations and the music from the special speakers within the cushion seem to soak deep into the body offering a truly profound sensory experience.

11. Safety

Please read all the Safety information below before each use

ALWAYS SUPERVISE THE USE OF THIS PRODUCT

- Use indoors, in dry environments.
- Keep cables safely out of reach.
- Remove all packaging and either store this in a safe place out of reach of clients or dispose of this responsibly. Plastic bags and wrapping can be hazardous to children and vulnerable people.
- Avoid sharp or pointed objects which could puncture the product. Take care with zips on clothing etc. Remove shoes.
- Do not place near sources of heat or fire that may damage the surface (radiators, fires, underfloor heating, hot central heating pipes etc.)
- This product contains a bead filling. The beads in the bead filling are small parts and should always remain out of reach. Do not attempt to access the interior part of this product. Never allow children or vulnerable people to access the interior part of this product.
- Should any seams or the zip become damaged, contact ROMPA® immediately to arrange repair or replacement. Should the product require re-filling, contact ROMPA® immediately to discuss this.
- This product contains a bead filling. The beads in the bead filling are small parts and should always remain out of reach. Do not attempt to access the interior part of this product. Never allow children or vulnerable people to access the interior part of this product.
- Some of our products contain zips for manufacturing purposes only. Do not attempt to open these or to access the interior part of these products. Never allow children or vulnerable people to access the interior part of this product.

12. Technical Specification

Weight: approximately 40kg

Speakers require a music system with:

Impedance: 6-8 OHMS

Minimum Amplifier/Music System Output – 50W RMS per channel

Maximum Amplifier/Music System Output – 200W RMS per channel


13. Installation

1. Place the Musical Positioning Cushion indoors, on a flat, stable surface. The zip around the PVC section should be on the floor. It is possible that some of the filling may have displaced slightly during transit. With use this will even out again, but you may wish to gently manipulate the cushion to make sure the filling covers the enclosed box. If you can still feel the integral box, check that the cushion is the right way up.
2. A 2-way speaker control is provided to allow selection of sound and vibrations either through the integral cushion speakers or external speakers, or both at the same time.
3. Insert the two leads from the integral speakers into SPEAKER # 1 OUT on the 2-way switch.


4. Insert leads from the external speakers into SPEAKER # 2 OUT on the 2-way switch.
 5. Insert leads from music system into SPEAKER INPUT on the 2-way switch.
 6. Plug in your music system.
 7. Select the required operation (CD/radio etc) and adjust volume to required level.
 8. SWITCH 1 on the 2-way switch operates the integral speakers in the cushion. SWITCH 2 on the 2-way switch operates the external speakers. Any variation may be selected.
- Please Note:** A high bass setting produces more vibration

PLEASE NOTE

Music Systems supplied approximately 2008 onwards may have different speaker connections compared with those of earlier models of music systems. Please refer to the diagrams below in case you have a newer model of music system. Older models of music systems tended to have small sockets with a round hole in that were covered by a small pushable cover (often red and black):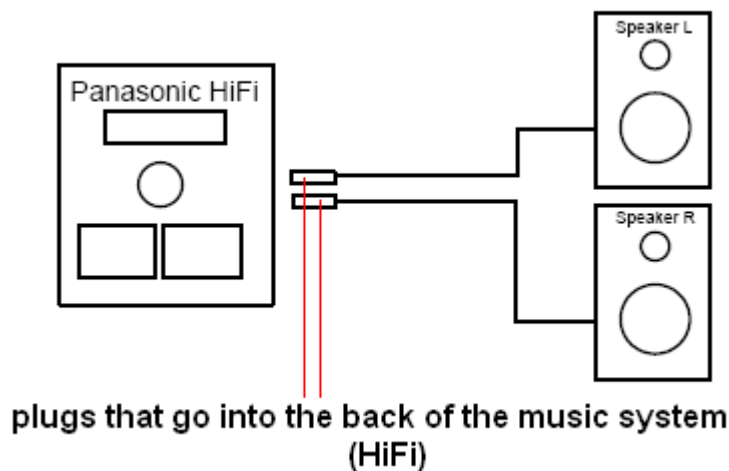

Please refer to the revised image below to show the connection of ROMPA® products to newer models of music systems:

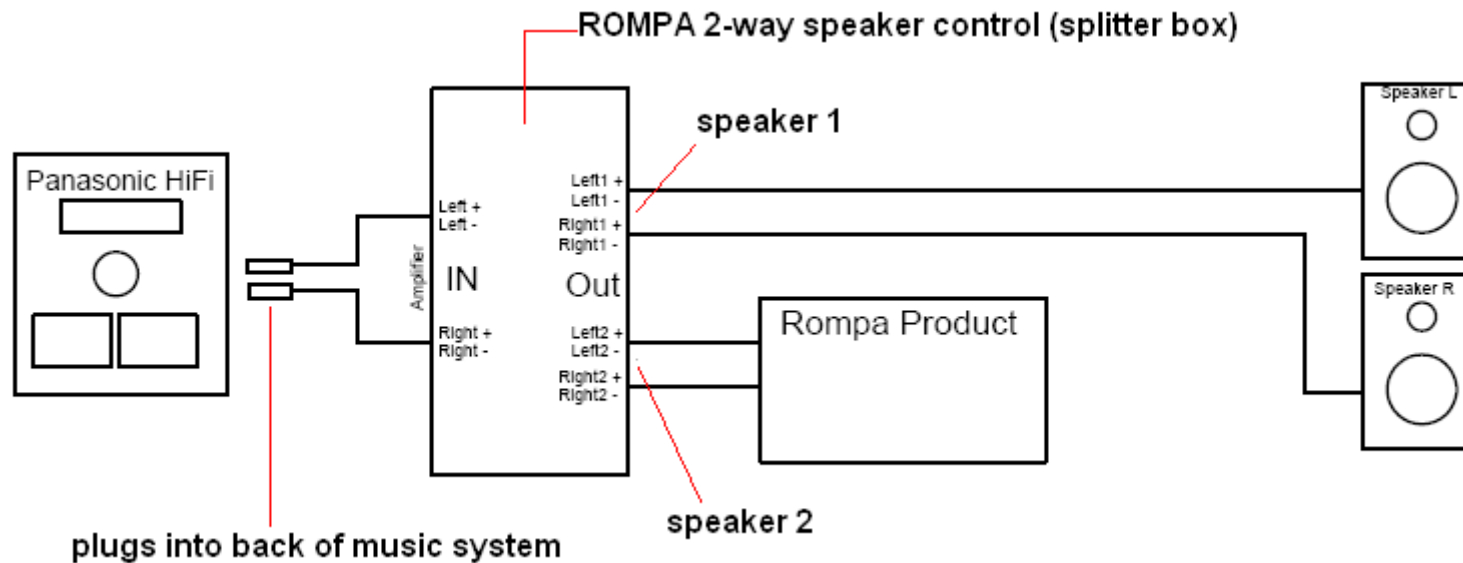If required, i.e. depending on your music system, it may be necessary to follow the steps outlined below:

1. Cut the speaker cables approximately 30cm from the plugs
2. Strip the ends of the wires attached to the plugs and connect to 'IN' of the ROMPA[®] 2-way speaker control (splitter box).
3. Strip the ends of the wires attached to the speakers and connect to 'OUT' of the ROMPA[®] 2-way speaker control (splitter box), Channel 1.
4. Connect the ROMPA[®] product (i.e. Musical Positioning Cushion) to 'OUT' of the ROMPA[®] 2-way speaker control (splitter box), Channel 2.
5. Use the switch on the ROMPA[®] 2-way speaker control (splitter box) to direct the sound to either: Speakers, ROMPA[®] product (i.e. Musical Positioning Cushion), or both.

14. Care and Maintenance

- For indoor use only.
- Always supervise the use of our products.
- Organise a regular cleaning schedule.
- Vinyl will crack, tear and rub off if subjected to sharp objects or continual rubbing by hard objects. Remove shoes, and avoid buckles, zips or sharp/hard/pointed objects coming into contact with the surface.
- Check seams regularly to ensure that these are not worn or split and the interior filling cannot be accessed.
- Wipe off accidental marks/liquids at the time they occur or are noticed.
- From time to time, if possible, lift floor cushion or move furniture to clean all surfaces and the floor underneath.
- Furniture and cushioning is intended to provide comfort and relaxation. Avoid eating and drinking whilst using these products.
- Do not place near sources of heat or fire that may damage the surface (radiators, fires, underfloor heating, hot central heating pipes etc.)
- Direct contact with bright sunlight may affect material over time.
- Do not saturate the material with lots of water.
- Some of our products contain zips for manufacturing purposes only. Do not attempt to open these or to access the interior part of these products. Never allow children or vulnerable people to access the interior part of this product.

PVC (Vinyl)

- Use a soft brush remove all loose dust/dirt
- Clean with a mild detergent (such as Lux soap flakes in luke-warm solution) applied to a sponge or damp cloth.
- This may be sanitised using a weak solution of Milton fluid.
- Do not use excessive amounts of water and always ensure the PVC is dry after cleaning.
- Do not use an abrasive cleaning pad.
- Do not use boiling/very hot water, washing up liquid / aggressive detergent or solvent based cleaner as this will cause the coating to discolour and go hard.
- Steam cleaning is not advised.

PU Nylon

- Use a soft brush remove all loose dust/dirt
- Clean with a mild detergent (such as Lux soap flakes in luke-warm solution) applied to a sponge or damp cloth.
- Do not use an abrasive cleaning pad.
- It is possible to steam clean this fabric.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

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