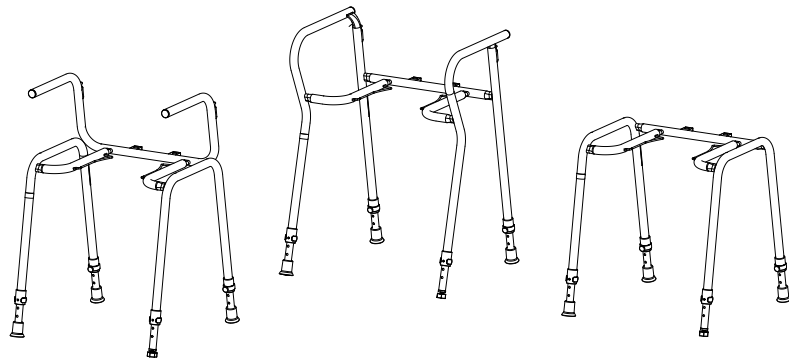
# Kombinera

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Shower and toilet chair frame

Art.nr: 811974013, 811974024,  
811974025, 811974064,  
811974065, 811974066,  
4065

## Beskrivning

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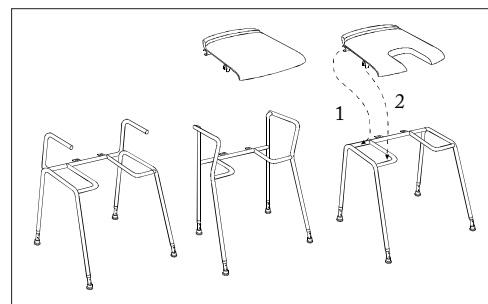
## Description

The Kombinera shower and toilet frame has been developed for use in the shower or as a freestanding raised toilet chair over the toilet or by the bedside. You can choose to equip the frame with a hygiene seat or a toilet seat for toilet use.

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
### Assembling the seat

1. Push the two holders at the rear, under the seat towards the frame and place the seat on the frame.
  2. Push with both hands on the front corners of the seat to put it in place.
- There are different kinds of frames; chair frames, stacking chair frames and stool frames. The seats are available in two versions; with and without hygiene recess.

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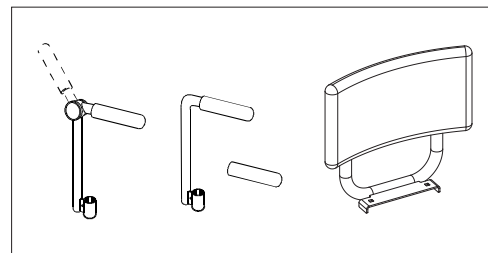## Accessories

When needed the frame can be equipped with different accessories:

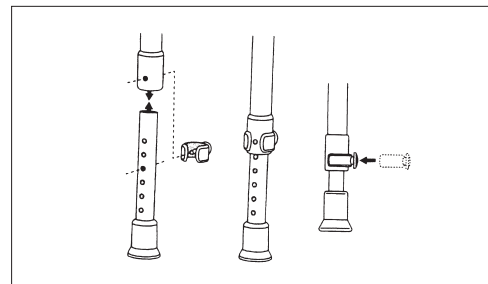
- flip-up arm rest (stool frames)
- armrest (stool frames)
- upholstered armrest (chair frames)
- backrest (all frames)

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## Height Adjustment

1. Remove the clip.
2. Pull out the leg for desired height.
3. Put the clip back into the hole.

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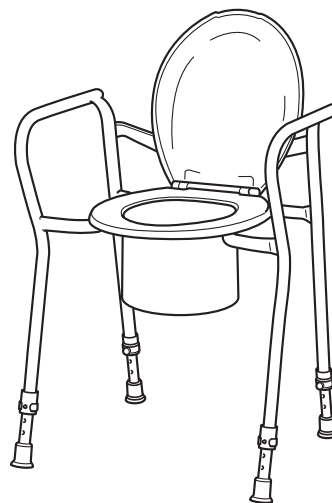
## Toilet chair:

Fit the toilet seat and lid using the holes at the back of the frame.  
Remember to check that the screws are fitted properly.

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## Accessories:

Splash guard: Used when Kombinera is placed over the toilet.

Bucket: For when Kombinera is used as a freestanding toilet chair by the bedside. Available in two versions, with or without handles.

Lid supporting strap: To provide support for the lid when it is in the 'up' position.

Safety strap: For when the user needs extra support to avoid falling forwards.

Roho toilet pad: If there is a risk of pressure sores, this Roho pad can be fitted to the toilet seat to achieve relief and reduce the risk of developing pressure sores.

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## Safety:

Warning: Do not angle the chair more than 7,5° (difference of 2 hole).

Max. weight: 160 kg. The chair must not be subjected to a load greater than 160 kg/353 lbs.

Cleaning: Clean the product using a solvent-free cleaning agent with a pH value of 5-9 or with a 70% disinfectant solution.

Decontamination: Can be decontaminated at max. 85° C/185°F for 3 minutes if required.

78391C

09-08-05

Etac, Box 203, SE-334 24 Anderstorp

Tel. +46/371-58 73 00, Fax. +46/371-58 73 90

www.etac.com

 etac®