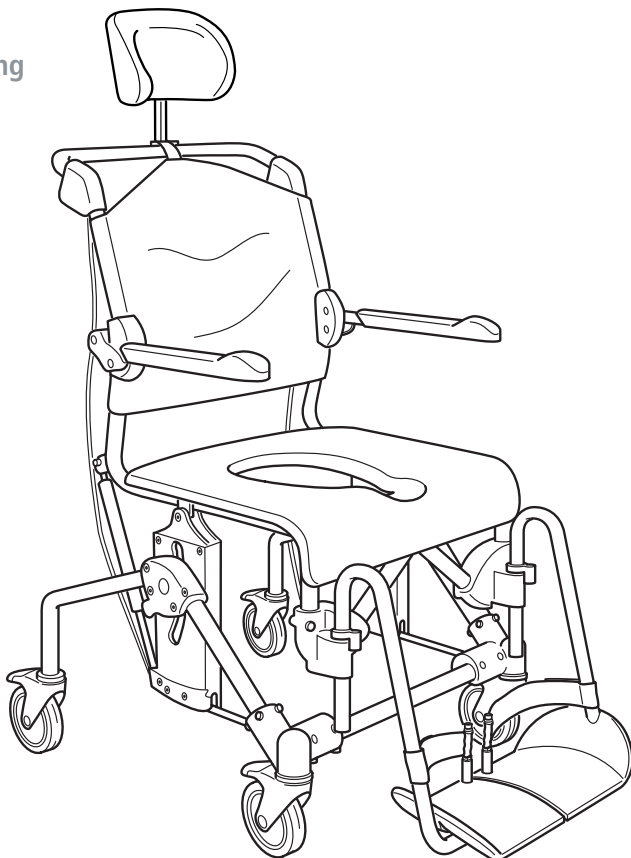
Swift Mobil Tilt

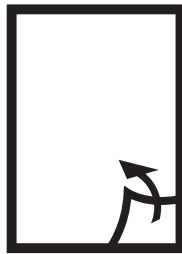
etac®

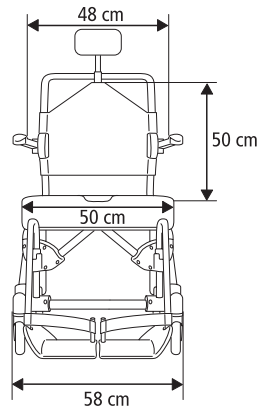
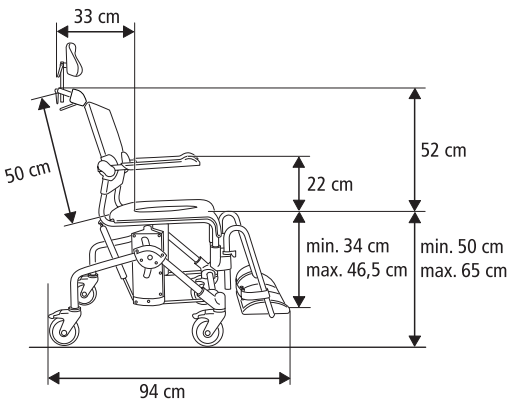
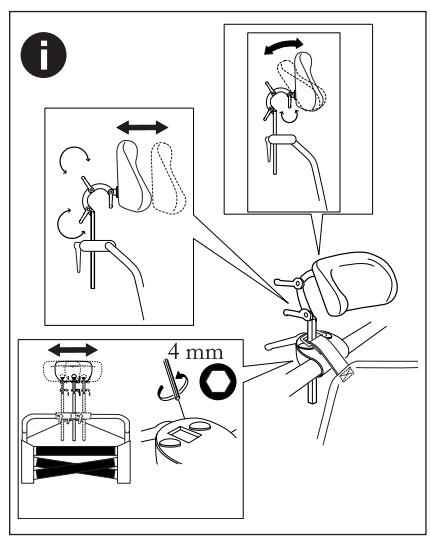
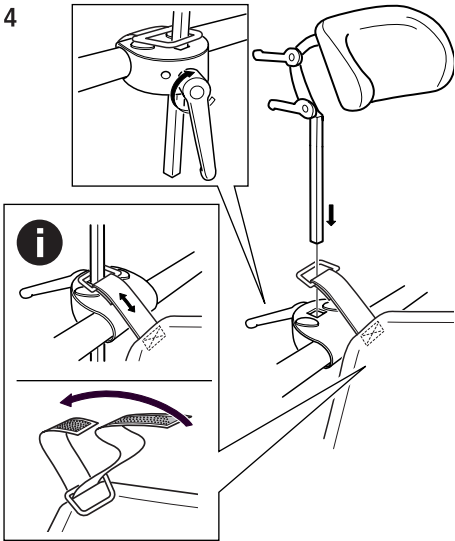
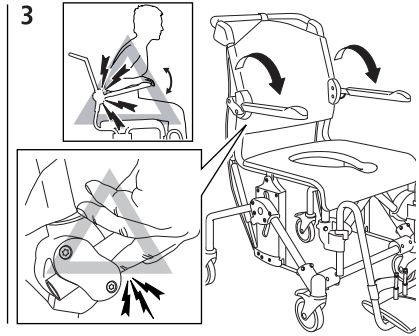
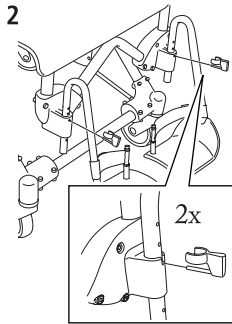
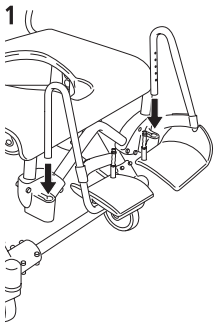
74886D 10-03-29








- SV Manual
- NO Bruksanvisning
- DA Betjenings-
vejledning
- FI Käyttöopas
- EN Manual
- DE Gebrauchsanweisung
- FR Manuel
- NL Handleiding
- IT Manuale
- ES Manual


A

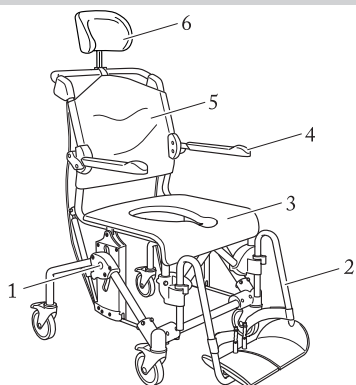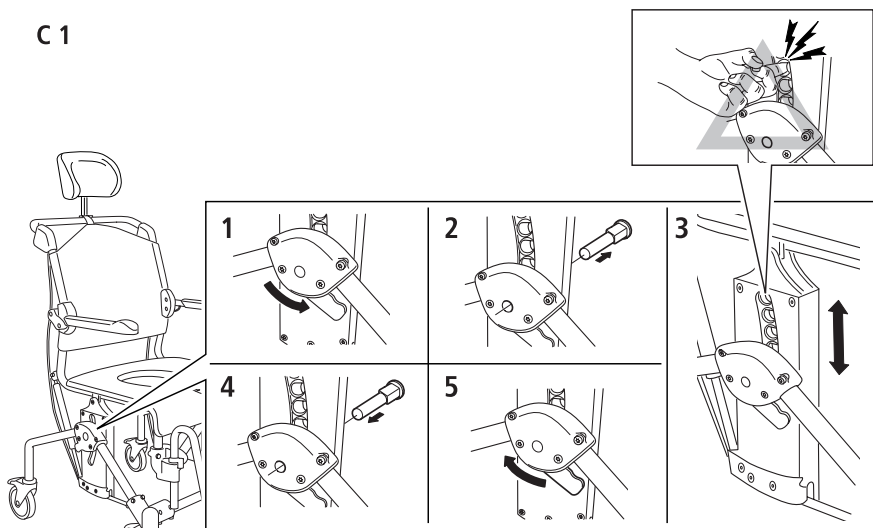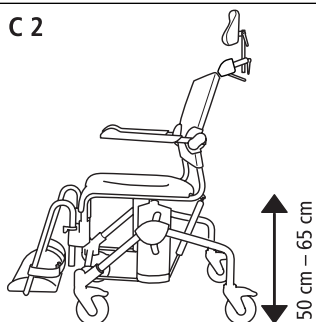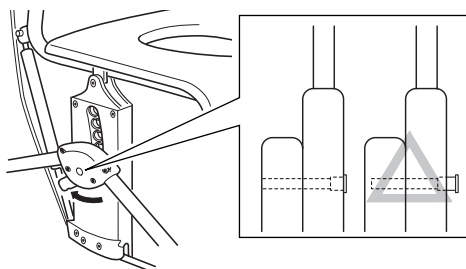B**C****C 1****C 2****C 3**

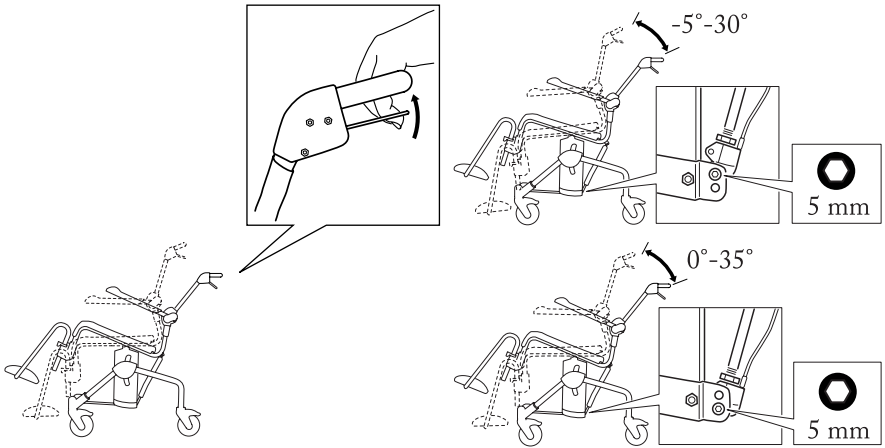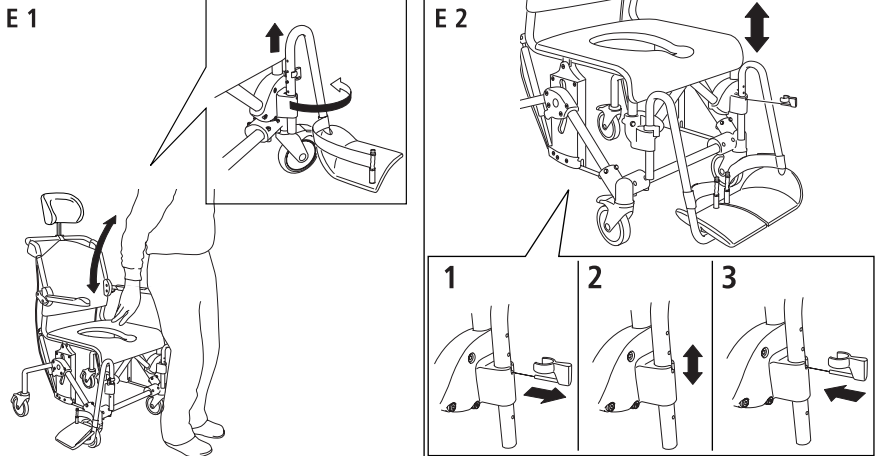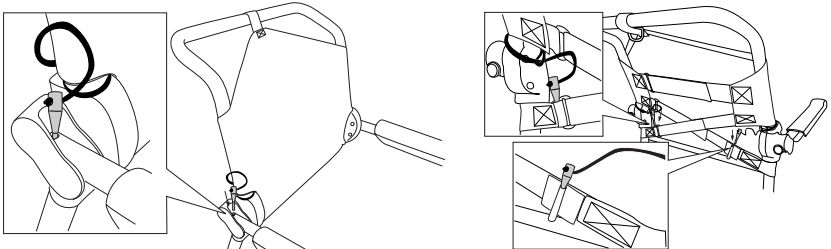
D**E****F**

G

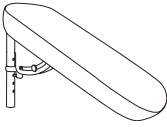
1

2

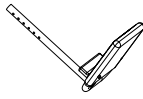3

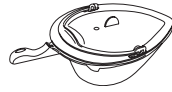
4


5

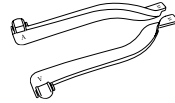6

7

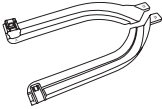8

9

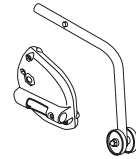
10


11

12


13

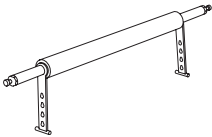
14

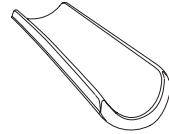15


16

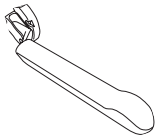
17

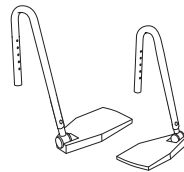
18

19

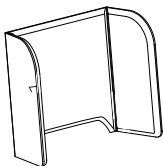20

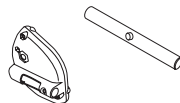SV Innehåll

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General information

Thank you for choosing a shower chair from Etac.

In order to avoid injury during handling and use, please read through this manual first. In the manual, the user is the person sitting in the chair. The assistant is the person operating the chair.

Swift Mobil Tilt is a shower and toilet chair intended to make hygiene care easier for people with reduced mobility and their carers.

Swift Mobil Tilt has been tested by a testing institute and complies with the requirements stated in EN ISO 10993-1 and SS EN 12182.

It is designed for use in showers, with hand basins and over toilets, as well as for transport to and from the sanitary room.

Every component of the chair is designed to make things easier for users and assistants.

This symbol appears in the manual along with text. It indicates instances where the safety of the user or the assistant may be at risk.

Assembly/Dimension table

fig. A

Product description

fig. B

1 Bolt	4 Armrest
2 Legrest	5 Backrest upholstery
3 Seat	6 Headrest

Height adjustment

fig. C

The height can be adjusted with ease by positioning the bolt in the required position (fig. C 1).

Swift Mobil Tilt is height-adjustable between 45 and 60 cm (fig. C 2).

Check that both sides are similarly adjusted, otherwise stability will be reduced. Do not use the chair if the bolt (orange) is not locked (fig. C 3).

Tilt function

fig. D

The assistant can adjust the sitting angle with ease by holding in the handle and tilting to the required position.

Make sure that the wheels are locked if the tilt function is used when the user is sitting in the chair.

Legrest

fig. E

The assistant can easily raise and swivel the legrests to the side for the user to sit down in or get up out of the chair (fig. E 1). The footrests are detachable.

Various height adjustment options are available by moving the locking pin (fig. E 2). The legrests have 6 different height settings.

Armrest lock (accessories)

fig. F

Secure the armrest in the lowered position by inserting the locking pin in the hole. When the armrest lock is not in use, the pins can be stored in pockets in the back of the backrest upholstery.

Recycling

This product should be recycled as per national regulations.

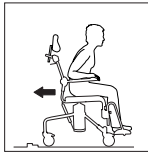
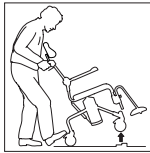
Transport

1. Make sure that at least two of the wheels are locked whenever the user is sitting down in or getting up out of the chair.
2. The legrests should be swivelled to the side whenever the user is sitting down or getting up.

Negotiating thresholds:

1. Chair only: Place one foot on the rear chassis tube.
2. With a user: move the chair backwards. Ensure that the castors are facing in the direction of travel.

Warranty/maintenance

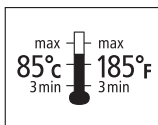
Warranty

The warranty period for material and manufacturing defects is 5 years.

Maintenance and reconditioning:

To keep the chair in good and safe working order, you should regularly:

- Clean the chair with ordinary non-abrasive detergent. Rinse and dry.
- The product can be cleaned at 85 °C for 3 minutes.
- Keep the castors free of hair and soap deposits, etc. If your shower and toilet chair is not in perfect working order, contact your dealer.

Safety

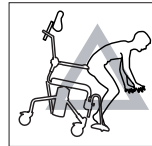
A defective chair should not be used.

Make sure that the wheels are locked whenever the user is sitting down in or getting up out of the chair and when using the tilt function with the user in the chair.

Anti-tip stabilisers should be used if the user is a leg amputee. Do not stand on the footrests, risk of tipping!

Swift Mobil can be used by people weighing a maximum of 135 kg.

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Etac Sverige AB
Etac AB (export)
Etac A/S