TOPRO BUTLER

Trillebord

Serving table

Servierwagen

TOPRO

Made in Norway

NO Bruksanvisning

EN User manual

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www.topro.no

CE

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TOPRO art.nr.: 814530

HMS art.nr.: 156464

Welcome as a user of the TOPRO BUTLER serving table

Every TOPRO product is developed, designed and made in Norway. Advanced manufacturing equipment, quality and environmental security give great accuracy and maximum quality in all the levels of production. The product is declared to be in conformity with Medical Device Directive 93/42/EEC. On receipt of the goods, please check your product. If you have any queries please contact your mobility shop or dealer immediately. The same applies if you have general questions about the product. If you have problems reading this user manual, an electronic version is available on www.topro.co.uk

Guarantee // Expected life span of the product

TOPRO BUTLER is guaranteed free for faults and defects for 2 years. Damage caused by incorrect use or parts that are exposed to natural wear and tear (e.g. wheels and table top) are exempt from the 2-year guarantee. For repairs during the guarantee period, please contact your local mobility shop or dealer. The guarantee is void if unauthorised spare parts and accessories have been or are being used on the product. The expected life span of the product is estimated to be 10 years, provided that it is used in the correct manner according to this user manual and its safety and maintenance instructions.

Technical data

Product weight	4,2 kg
Maximum load	7 kg
Width	540 mm
Depth	420 mm
Height handles	710–1020 mm
Height table top	490–800 mm
Table top width × depth	470 × 360 mm
Diameter wheels	80 mm
Width on wheels	25 mm
TOPRO art.no.	814041

Intended use

TOPRO BUTLER is designed and approved for indoor use as a serving table to help carry around items like kitchen tools, food, clothing etc. For easier use, and to reduce the risk of unnecessary strain, the curved aluminium pipes, which makes the handlebar of the product, is designed to give the user alternative grips and heights adjustments. The TOPRO BUTLER must never be used as walking aid.

Important to know about safety before use

- Take care when assembling; do not trap your fingers.
- Make sure the plastic nuts for the height adjustment of the table legs are firmly tightened.
- Be aware that some parts of the serving table might feel hot or cold when exposed to extreme temperatures.
- Do not alter the product as this might put your safety at risk and the guarantee will be void.

Important to know about safety when in use

- Maximum load of the serving table is 7 kg.
- Do not use the serving table to transport heavy loads or persons. Do not sit on the table.
- The table top can endure rough treatment like e.g. a warm kettle. However, do not put items on the table top that are at boiling point temperature, such as pans from the cooker etc.
- The serving table is not a toy and therefore should be treated in an appropriate manner.
- Be careful when crossing carpet edges, thresholds etc. For safety reasons it is recommended to lift the serving table/wheels nearest the carpet edge/threshold and then pull the serving table over.
- Ensure the parking brakes on all four wheels are released before pushing the serving table.
- Do not use the serving table on stairs.
- Do not push the serving table too far from your body as this can cause unnecessary strain and lead you to fall.

Height adjustment

The height can easily be adjusted without tools. Unscrew the plastic nut of one of the table legs and adjust the pipe to a suitable height. Tighten the plastic nut. Adjust the other table legs accordingly.

Parking brake

The TOPRO BUTLER has parking brakes on all four wheels. Step on the pedal to lock the wheel. To release, pull the pedal up again.

Storage

The serving table should be stored indoors in an upright position.

Material // Recycling

The frame of the serving table is made of plastic coated aluminium pipes, the couplings and table top of thermoplastic material. All parts can be recycled. Dispose the serving table according to the applicable regulations in your country. For information, please contact your public authorities.

Cleaning

The serving table can be steamed, hosed and washed with normal household detergents. Do not use scrub detergents, steam- or high-pressure cleaners.

Disinfection

Disinfection must be carried out by authorized personnel only and by wearing sufficient protective equipment. The surface of the serving table must be cleaned with a disinfection that contains 70–80% ethanol. We advise against disinfection that contains chlorine and phenol.

The manufacturer cannot be held responsible for any damage or wounds that might occur through the use of harmful disinfection or disinfection carried out by unauthorized personnel.

Inspection // Maintenance // Re-use

It is recommended that general inspection and maintenance is carried out and that the items below are checked on a regular basis (how frequent is determined by how and how often the serving table is used):

- Plastic nuts for height adjustment of the table legs: Are they firmly tightened?
- Table top: Is it firmly attached and free of cracks?
- Wheels and parking brakes: Are they working properly and not covered with filth, dust etc.? Are the wheels properly attached?
- Pipes: Are they without damages?

The items above also applies when the serving table is prepared for re-use.

Both table top and wheels are available as spare parts.

Table top art.no.: 814637

Wheel art.no.: 814632

Accessories (to be ordered separately)

Basket

Easily assembled and dismantled without any use of tools.

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