

Equipment	Cleaning Instructions
Lateral Supports	Wipe clean with sterile wipes or hot soapy water
Pillows	Wash to 40°
Sheets	Wash at 40°. Can be washed to 60°
Top-to-Toe Mesh	Wash to 40° (steri-touch mesh combats MRSA AND C-Difficile anyway)
Supine Stabilizer	Wash to 40° with outer cover on. Wash to 60° without outer cover on. Wash to 90° for heavy soiling without outer cover (if desperate)
Mattress Topper	Wash to 95°
Rolls and Wedges	Wash to 40°. Can be washed to 60°
Lateral Supports Padded Covers	Wash to 90°
Foot Supports	Wash to 40°. Can be washed to 60°