



Quick start installation guide for a cushion loop

Be aware of that if you only need to use the cushion loop, then it isn't necessary to use all the items in this TV-kit package

1 Plug the connector cable to the SCART adaptor and connect it to the television (some models do not need a SCART adaptor, In that case plug directly into TV). Then plug the jack "a" into Radio/TV on the Field box

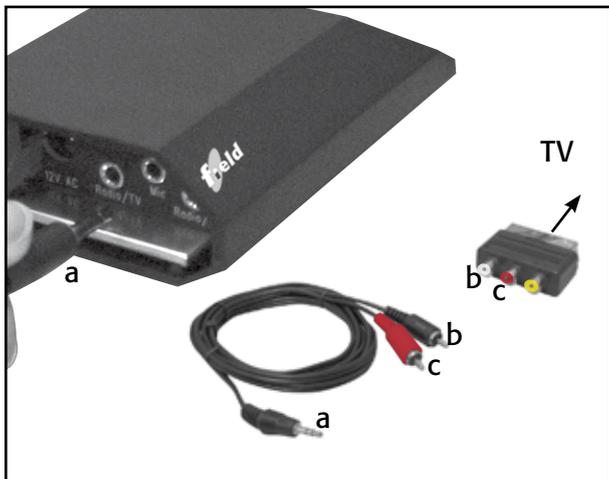


2 Plug the transformer into a wall socket and into the power connection on the amplifier

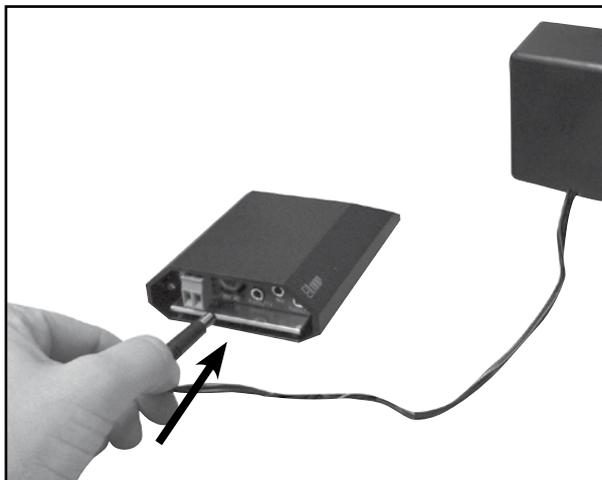

3 Place cushion loop pad under a cushion in the seat where you will be listening to the sound source



4 Remove the green jack from the field box. Save it for future use for the teleloop cable

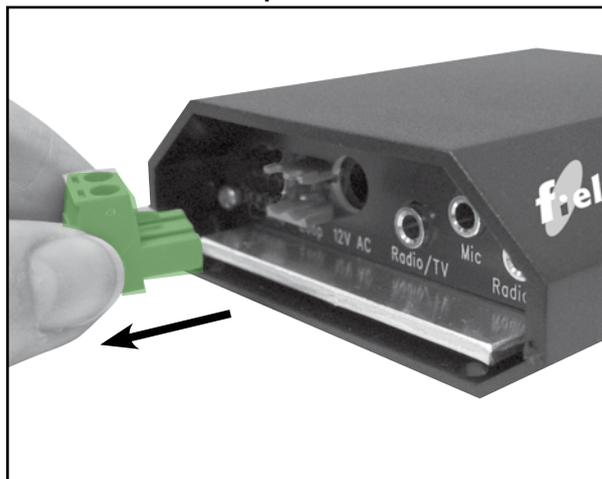

5 Connect the cushion loop to the field box



6 Set hearing aid to T-position and adjust volume



International +45 3917 7101

))))))PhonicEar®

Note: Use only the cables and jack supplied by an authorized Phonic Ear representative/installer or the product may be damaged.

921-34-310-00/Rev B/0510