


Attainment Company's

GoTalk[®] 32+ *user's guide*

Recording time:
10 seconds per message
17 seconds per core vocabulary

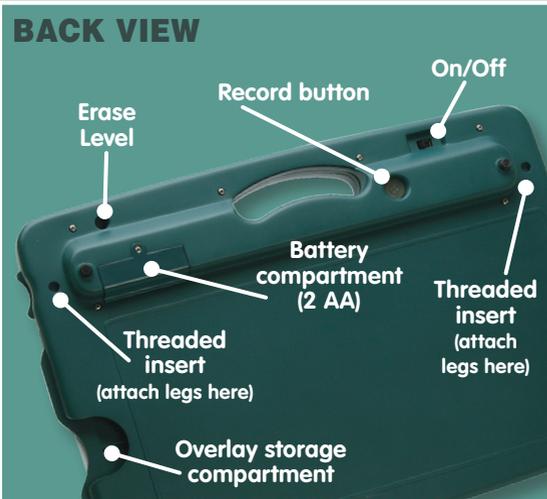

Weight: 2 lb. 1 oz. (.95 kg)
Size: 14.25" x 10" x 1.625" (36.2 cm x 25.4 cm x 4.1275 cm)
Total recording time: over 27 minutes

GoTalk 32+

The GoTalk 32+ is a voice output communication device with multiple messages and levels. You decide the statement, word, or sound to be recorded on each message key, and the image or word that will represent each message. Print the overlay (grid with images or words) and record the messages: GoTalk 32+ is ready to go!

Levels

The GoTalk 32+ has five levels; each level stores its own set of 32 messages. The level change button is on the front, to the right of the handle. The five numbered green LEDs represent the levels. To change levels, press and release the green level button. A green LED indicating the current level will light. Press and release the button again to move to the next level.

A Level Lock feature is available to prevent unwanted changing of the levels. To lock the device onto a specific level, see **Lock Options** for details.

Volume

Volume up and volume down buttons are located on the front, to the left of the handle. A beep will sound when adjusting the volume up or down. The beep will not sound when the volume has reached the maximum or minimum setting.

Core Vocabulary Messages

Messages recorded on the Core Vocabulary keys will play the same message on every level; they are not connected to a specific level. Each of these three messages have 17 seconds of record time. The overlay strip for the core vocabulary messages is separate from the overlay for the grid of 32 messages. To record or re-record one or all three Core Vocabulary, follow **Record Messages** instructions.

Record Messages

Messages have 10 seconds of record time. Turn on the GoTalk 32+: all LEDs cycle through once, device beeps, a green LED flashes indicating the level currently in use. To record, press and release the Record button (on the back); a Level LED (green) and the Record LED (red) will light. While both LEDs are lit, press and release the message key to be recorded on; the Record LED blinks to indicate the device is recording. Speak clearly while recording, then press any key to stop recording. Continue to record additional messages or exit Record mode. To exit Record mode, press and release the Record button: both LEDs turn off. A Record Lock feature is available to prevent unwanted recording or erasing of messages. To lock the recording ability, see **Lock Options** for details. Also, see Tips and FAQs for recording tips.

Play Messages

Press and release a message button; a green level LED will blink while the recording plays.



Erase Recordings

To erase individual message keys, replace the message with another recording.

Turn on Record mode. Press and release the message key to start recording. Then quickly press any key to stop recording (tap the message key).

To erase all recordings on a level, select the level to be erased (Level button). Turn on Record mode. Press a pen point or paperclip into the Erase Level pinhole on the back of the GoTalk 32+. The red Record LED will stay lit and the green Level LED will turn off. All messages on this Level will be cleared. Erase level does not erase core vocabulary.

Lock Options

Prevent unwanted use of Record and/or Level buttons

Level Lock—To turn On Level Lock, press and hold the Record button. The red Record LED and green Level LED will turn solid. While pressing the Record button, press and release the Volume Down button. The Record LED and the Level LED will turn off indicating Level Lock is ON.

Record Lock—To turn On Record Lock, press and hold the Record button. The red Record LED and the green Level LED will turn solid. While pressing the Record button, press and release the Volume Up button. The Record LED and the Level LED will turn off, indicating Record Lock is ON.

To **UNDO** the Record and Level Lock, turn the GoTalk off. While off, press and hold the Record button and turn the unit On. Once the device beeps the Record button can be released. The Locks now are turned off.

Legs

Two threaded post legs with rubber tips are included. Using the legs will add a slight slant to the device; legs are optional while using the GoTalk. Screw the legs into the matching thread holes on the back of the device.

Batteries

The GoTalk 32+ uses two AA batteries (included).

Overlay Storage Compartment

Overlays for levels not currently in use can be stored in the compartment on the back of the device.



Tips and FAQs:

- During the record process you can listen to the last message recorded without exiting Record Mode by pressing the Volume Up or Volume Down button.
- Problems changing levels? Level Lock might be turned On. See “UNDO” paragraph under “Lock Options.”
- Problems recording? Record Lock might be turned On. See “UNDO” paragraph under “Lock Options.”
- **Recording tip:** record in a quiet environment. The GoTalk microphone will pick up and record all sounds, even unintended noises. The microphone is located in the upper right corner, next to the Record (red) LED.
- **Recording tip:** set the GoTalk on a flat surface while recording. Moving your hands on the device can cause unwanted sounds to be recorded. Also, press the message button gently to start and end recordings. The device is sensitive and requires very little pressure to activate.
- **Recording tip:** to achieve a more fluent communication style, avoid empty recorded time. Each message key has a set amount of time designated to it. Often messages to be recorded are much shorter than the time designated. Begin recording message as soon as the device starts recording and stop recording process immediately after message is spoken. Do not wait for allotted time to expire.

GoTalk™ Overlay Software

Creating overlays is easy with GoTalk Overlay Software! Includes over 4500 Imagine Symbols and photos. You can also use an integrated online image search feature or use your own digital photos and images in any overlay. Overlay cells can contain multiple images and text boxes. Editing features let you change color, size, and font of text; move, enlarge, rotate, and crop pictures; and export files in PDF, JPG, or PNG formats so it's easy to email overlays to others. Includes dozens of ready-made templates for all of Attainment's AAC products.



available at www.AttainmentCompany.com

Go Talk 32+ accessories



Attainment Company, Inc.

P.O. Box 930160 • Verona, WI 53593-0160 • USA • Phone 1-800-327-4269
International Calls +1-608-845-7880

www.AttainmentCompany.com