

1 Insert Battery and Turn On

1. Remove phillips screw and battery door.
2. Insert one 9-V battery and re-attach battery door and phillips screw.
3. Slide power switch to on.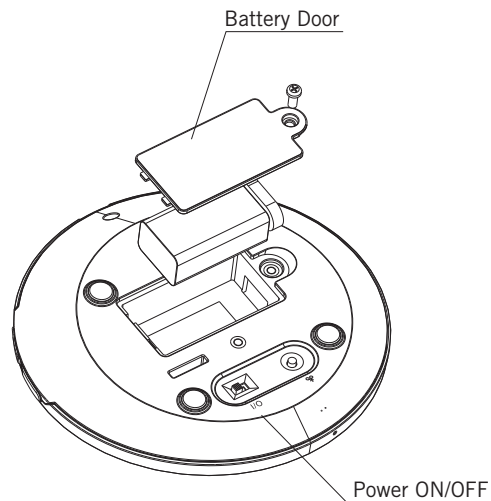

2 Record A Message

1. Press record button and the visual feedback light will turn red.
2. Touch the activation area. When you hear the auditory beep and the visual feedback light starts to blink, begin recording your message into the microphone.
3. When you are finished recording, press the record button again.

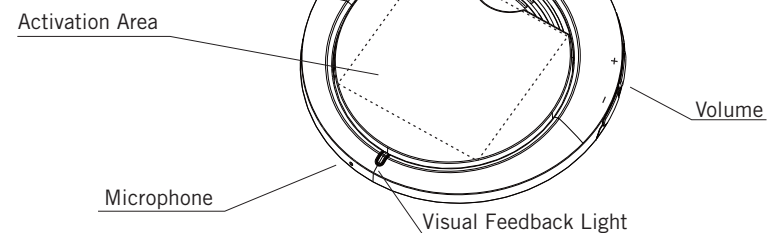3 Playing A Message

1. Touch the activation area to play message.
2. Adjust volume up or down as needed.
3. You will see the visual feedback light turn on when the activation surface is touched and message plays.

5 Low Battery Indicator

The following are indicators of a low battery and the battery should be replaced.

1. Immediately after a message plays, the visual feedback light blinks red three times.
2. A message begins to play, but stops before completely playing and the visual feedback light blinks red three times after the message stops playing.

4 Erase A Message

1. To erase a message either record a new message over the existing message or hold the record button for three seconds. You will hear an auditory beep when the message has been erased.

Contact & Warranty Info

Warranty/Repair/Return Information

AbleNet offers a limited two-year warranty on QuickTalker communication products. For full warranty details, visit www.ablenetinc.com.

For more information on AbleNet products and services.

AbleNet, Inc
Minneapolis/St Paul, MN
800-322-0956 (US & Canada) / www.ablenetinc.com
© 2012 AbleNet, Inc., All Rights Reserved.

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© 2012 AbleNet, Inc., Todos los derechos reservados.
800-322-0956 (U.S.A et Canada) / www.ablenetinc.com
© 2012 AbleNet, Inc., tous droits réservés.