ACTIVE *Sinnflut*®

Man

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Personal Massager for Men

rechargeable

Instructions for use

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Rechargeable

Mode d'emploi

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Gjenoppladbar

Bruksanvisning

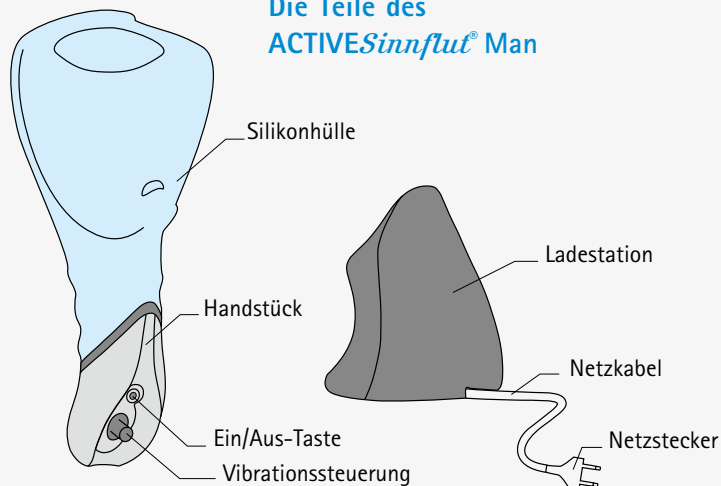
Deutsch
English
Français
Norsk

MED*intim*

personal healthcare

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Einleitung

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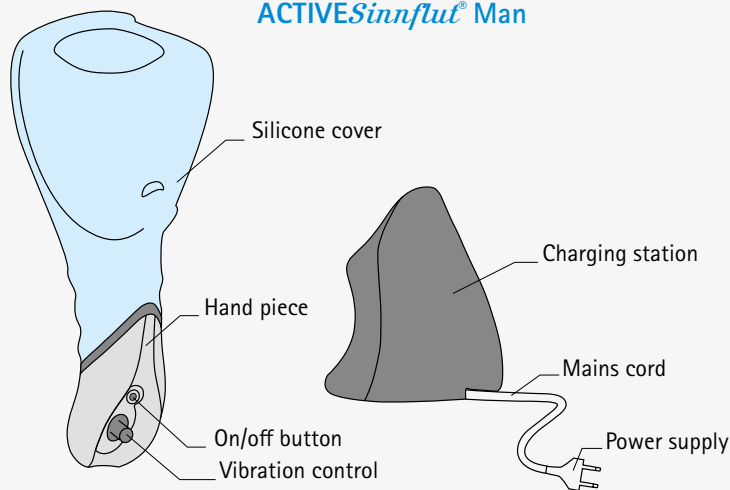
ACTIVE*Sinnflut*® Man

Introduction

We congratulate you on the purchase of your **ACTIVE*Sinnflut*® Man** personal massager with charging station and we wish you great pleasure. In order to guarantee optimum results and safe operation, we ask that you:

- Start by reading the operating instructions carefully and observe all safety information.
- Use the personal massager only in such a way, as it is described in the operating instruction.
- Keep the operating instructions for future reference.

If you still have questions after reading the operating instructions, please get in touch with our customer service representatives or visit our website www.medintim.de

Packing

The equipment **ACTIVE*Sinnflut*® Man** is packaged to protect against shipping damage. The packaging consists of recyclable raw materials.

KEEP THE OPERATING INSTRUCTIONS FOR FUTURE REFERENCE.

Important notes

During development of **ACTIVE*Sinnflut*® Man** products, special care has been taken to ensure high quality craftsmanship and user safety. However precautionary measures should always be observed when using any electrical device.

Precautions against dangerous electrical shock

- Do not place the charging station over or near bathtubs, showers or washbasins, into which it could fall.
- Do not dip the charging station, cables and/or plug into water or other liquids.
- Do not reach for a charging station that has fallen in the water, but immediately unplug the station from the power supply.
- Do not plug the charging station into an electrical socket with incorrect voltage. Only plug into sockets that correspond to the voltage levels indicated on the identification plate found on the bottom of the charging station. Voltage regulators and adapters do not ensure compliance with product electrical requirements.
- Do not force the power supply plug into an electrical socket.
- Keep electrical appliances away from children.

Precautions against burn, electrical shock, fire or other injury

Use this product in accordance with the conditions in these operating instructions and do not use accessories that were not recommended by the manufacturer.

- The charging station and personal massager must not come in contact with hot surfaces.
- Do not use the equipment in areas where there is a high risk of explosion.
- Never open the charging station on your own. Inappropriate handling may result in injury.
- Do not remove the power supply plug from the electrical socket by pulling on the charging station's mains cord, This can damage the cord.
- Do not use the charging station, if the mains cord or the power supply plug is damaged, if the appliance is not working properly, or after it has been dipped into water.
- If the mains cord is damaged, it must be repaired in order to avoid possible injury. Repairs should only be made by our service technicians or a similarly qualified person.

Choice of location for the charging station

- Do not place the charging station in a humid room or near a water-filled Container.
- Place the charging station on a horizontal, even, no-skid surface to avoid tipping.
- Select the location on an electrical socket, which is within reach of the charging station's 1.40 meter mains cord.
- Normally the feet of the charging station do not leave imprints on the storage surface, however this can not be completely excluded.
- Avoid excessive smoke, dust, vibration, chemicals, humidity, heat or direct solar radiation.

Medical references

The materials used for the personal massager conform to the guidelines of the BfR (German federal institute for risk evaluation) and the FDA (US Food and Drug administration).

- Do not use a personal massager with damaged or torn silicone cover.
- The **ACTIVESinnflut® Man** personal massager serves exclusively for massage and stimulation. Do not use this device for other purposes.
- If a surgical intervention has been performed on the concerned body parts, ask your physician before using the personal massager.
- Contact your physician, if problems arise after use.
- Electrical devices functioning within close proximity can influence each other. This appliance should not be used within one meter of medical instruments or equipment.
- Carriers of pacemakers should address their questions about the use of this appliance to their physician, **ACTIVESinnflut® Man** products should not have an adverse impact on most commercially available pacemakers.
- The personal massager **ACTIVESinnflut® Man** is intended for the individual use. If this appliance is used by more than one person, the use of a condom is urgently recommended as a protective layer over the penis to protect against possible transmission of sexual transmitted infections.

Use

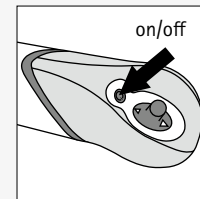
Avoid bending or a rotating of the silicone cover. This may cause damage to the silicone cover or the Vibration elements. Use the personal massager **ACTIVE*Sinnflut*[®] Man** exclusively with water-based lubricants. Use of oily silicone lubricants for long periods of time can lead to permanent surface changes in the silicone cover.

To extend the life of the battery, the **ACTIVE*Sinnflut*[®] Man** personal massager comes with discharge protection that automatically switches off the appliance when the minimum battery capacity is reached. Before automatic shut down, short interruptions in Vibration indicate that you are approaching minimum battery capacity. Recharge the personal massager in the charging station after the automatic shutdown.


The personal massager **ACTIVE*Sinnflut*[®] Man** can be cooled or warmed up before use. Do not exceed the temperature range of 0° to +40° Celsius. The capacity of the battery decreases as temperature decreases. Avoid sudden temperature changes.

Start-up and control


When manipulating the on/off key and control pin, switch them only to the noticeable pressure point and the perceptible change of vibration. Excessive effort on the controls can damage the personal massager.

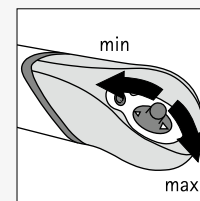
Press the on/off key on the personal massager to switch it on. The intensity of vibration rises briefly to maximum strength after switching the personal massager on and then drops back to the medium intensity.

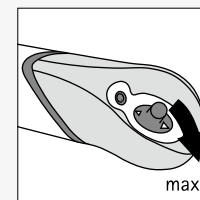To increase of the strength of vibration press the control pin toward "max". To decrease vibration, press it toward "min". With individual pressure impulses the vibration changes gradually through all 9 degrees of intensity. If you keep the control pin pressed longer than 1 second, the vibration strength changes independently.

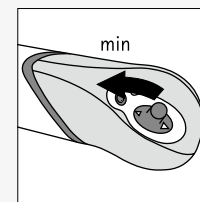
To run one of the 3 vibration programs, leave the control at maximum vibration strength for 3 seconds then press the control towards maximum again, until you feel the vibration change. You can change the program by pressing the control towards "max" again.


To leave the vibration programs, press the control toward "min".

Charging

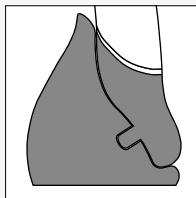The **ACTIVE*Sinnflut***[®] **Man** personal massager comes with charged batteries. Before charging the personal massager for the first time, allow it to completely discharge and then charge the device for at least 10 hours. The maximum efficiency of the batteries is only reached after five complete charges and discharges.

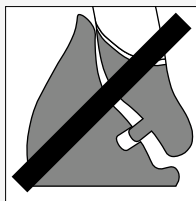Place the personal massager into the charging station with the hand piece and the vibration control cone in front. The personal massager must sit precisely in the charging station to ensure the transfer of current.

The batteries are not charged if the personal massager is not correctly positioned.

The personal massager aligns itself automatically when inserting it into the charging station. In order to avoid damaging the equipment, do not force the hand piece into the charging station. Heating up of the device during charging is normal and harmless. It is not possible to overcharge it.

We recommend keeping the **ACTIVE*Sinnflut***[®] **Man** personal massager in its charging station, so that the batteries are fully loaded all the time. Complete charging of the batteries takes at least 10 hours. The charging procedure must be repeated at least every 3 months in order to obtain the maximum capacity of the batteries and to ensure a long lifespan.

Cleaning and maintenance

Clean the **ACTIVE*Sinnflut***[®] **Man** personal massager before and after each use. We recommend the cleaning with mild soap under flowing, lukewarm water. When cleaning, do not use sharp or pointed articles, corrosive cleaning agents or hot water exceeding +40° Celsius, which will damage the personal massager. Dry the personal massager completely, before you place it on the charging station. Clean the charging station regularly. Before cleaning, disconnect the charging station from the power supply. For cleaning we recommend mild soap and a dampened cloth.

The charging station must not be exposed to high humidity or wetness under any circumstances. Dry the charging station completely, before you reconnect the appliance to the power supply.

If you want to disinfect the personal massager or charging station, contact your physician or pharmacist about the most suitable means.

Examine in regular intervals the mains cord. If the mains cord is damaged, do not use the charging station.

Environmental considerations

The personal massager contains a nickel metal hydride battery pack. The battery pack cannot be exchanged, but may be taken out for the purpose of the waste separation.

There are containers for GRS batteries (common collection system) as well as collection places for the normal disposal of battery packs.

With the removal of the batteries the warranty expires and the appliance can no longer be repaired.

Replacement parts

All **ACTIVESinnflut® Man** personal massagers and charging stations are compatible with each other. Both components can be bought individually. Please ask our customer services, or visit our website: www.medintim.de.

Trouble shooting

If you cannot solve a problem using the following reference, ask **ACTIVESinnflut® Man** customer service or visit our website www.medintim.de.

The personal massager **ACTIVESinnflut® Man** does not function.

- Load the personal massager in the charging station for at least 10 hours.
- Make sure that the charging station is connected to a live electrical socket and that the socket has not been switched off with the space light.

The vibration seems to be limited to low power.

- Check the vibration strength setting.
- Ensure that the personal massager has not been stored without being used for a long period of time, at low temperature or off of the charging station thereby exhausting the charge capacity.

There are interruptions in the vibration.

- Check that the personal massager is not set to a vibration program.
- The charge capacity of the batteries is nearly exhausted and recharging will be necessary.

Unpleasant smell

- Clean the personal massager and the charging station carefully.

Technical data

Charging station: Power supply 230 V, 50 Hz

Vibrator: Power supply 2 x 1,2 V 1 200 mAh Ni-MH

Power: 2 W

Charging time: 10 h

Hours of operation at medium vibration intensity: 2 h

Allowable operating temperature: 0° to +40° C

Warranty

ACTIVESinnflut® products are manufactured with great care and under constant production control. If a material or manufacturing defect should occur within 12 months after the date of sale, this appliance can be sent to the distributors address provided. In order to accelerate the process, first contact **ACTIVESinnflut®** customer service. Please keep your receipt as proof of purchase.

The warranty does not cover damage resulting from any of the following:

- Inappropriate use
- Use of oily silicone lubricants
- Non-compliance with cleaning instructions
- Non-compliance of the charging notes for the battery pack
- Unauthorised interferences
- Excessive forcing

For repair, send the appliance thoroughly cleaned and appropriately packed together with a description of the problem, your address and the receipt to the distributor address. (Freight forward consignments cannot be accepted)

Shipping damage is not covered by warranty.

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KESSEL medintim GmbH