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I tvivl?: +45 8693 8585

## Washing- and tumble instructions

Washing temperature: **40 - 85°C**  
Tumble dry: **max 60°C**  
Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds

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UK

### WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

### IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately
- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the toppiece

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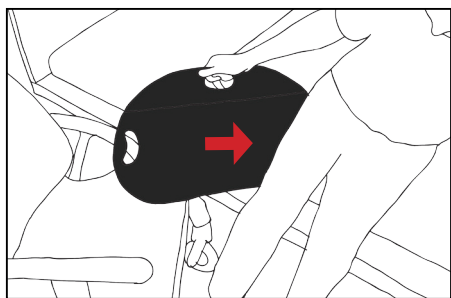
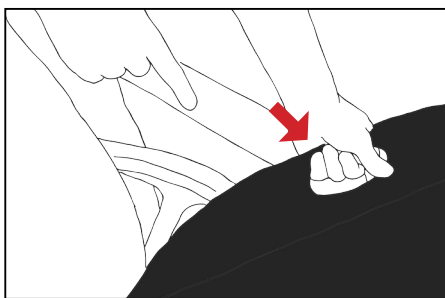
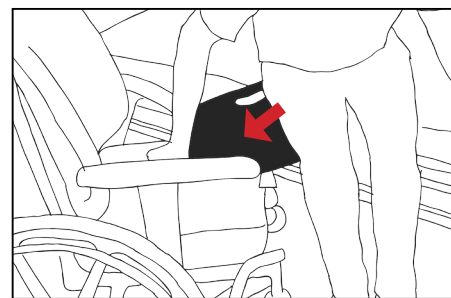
**UK** The patient leans a little toward the hip facing the main body of the bed. The patient takes hold of the central handle of the Master Glide and pushes the glide board under the buttock facing the wheelchair

So that the patient does not bump into the wheel while being transferred, bend the rear edge of the Master Glide up, thus shielding the wheelchair wheels.

**NB!** Model Junior, Senior and Heavy do not have this function

The patient leans against the glide board, thereby putting the weight of the buttock on the glide board. The patient grips the handle of the wheelchair with his/her free hand and, using the Master Glide, slides over to the wheelchair seat

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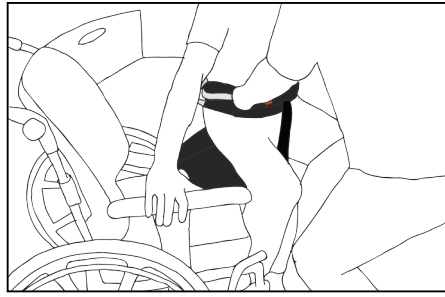
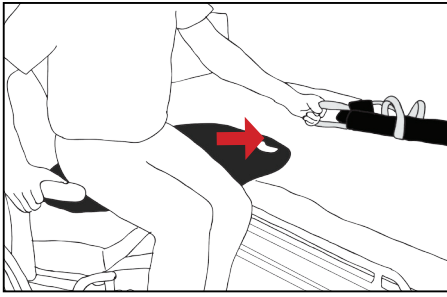
**UK** To remove the Master Glide again, the patient grips both handles on the glide board and bends it up. The anti-slip coating on the glide board is lifted up from the surface. The glide board is kept tight and pulled out from under the seat portion

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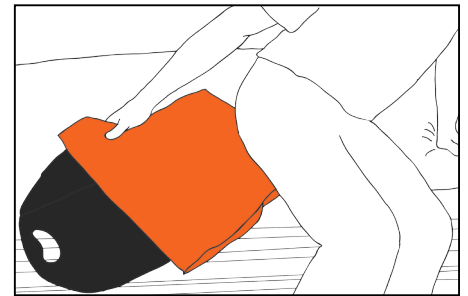
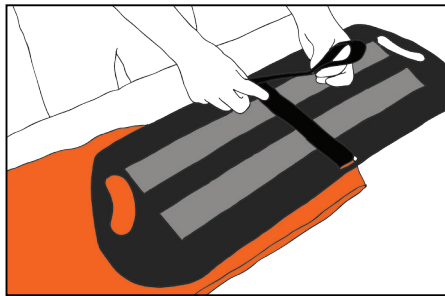
**UK** In order to provide peace of mind and activation, the patient can use the Master Bed Grip to pull his/herself over in the bed. The bed tape is attached to the bed

To help with the transfer while supporting the patient, the Master Belt can be beneficially used. This is attached to the patient's waist. The helper grips the handle and supports against the patient's knees. The patient is transferred to the wheelchair

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**UK** When moving a patient with bare skin, use the Master Glide sliding piece. The velcro tape on the slider is opened and the velcro tape is fed down into the two tracks on the Master Glide Flexi from the surface and downward

The ends of the velcro tape are brought together and locked. The sliding piece is mounted onto the Master Glide Flexi's centre

Place the Master Glide Flexi under the patient's buttock as the patient tilts toward the opposite side. The loose sliding piece is pushed in under the patient's lifted buttock. The patient can now slide, without friction on his/her skin

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UK

- Specially made for children
- Allows transfers over distances of up to 20 cm
- Easy to position and to remove
- Enables mobilisation of the user
- Can be used both independently and with an assistant's help
- Low weight
- Easy to clean
- Can be hung up

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UK

- Allows transfers over distance of up to 20 cm
- Easy to position and to remove
- Enables mobilisation of the user
- Can be used both independently and with an assistant's help
- It is shaped so there is room for wheelchair wheels
- Low weight
- Easy to clean
- Can be hung up

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UK

- Allows transfers over distances of up to 20 cm
- Easy to position and to remove
- Enables mobilisation of the user
- Can be used both independently and with an assistant's help
- Mounting the slide piece allows transfers with bare skin
- Has a flexible rear edge for protection from the wheelchair wheels
- Low weight
- Easy to clean
- Can be hung up

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UK

- Heavy Small allows transfers over distance of up to 20 cm
- Heavy Medium & Large allows transfers over distance of up to 40 cm
- Easy to position and to remove
- Enables mobilisation of the user
- Can be used both independently and with an assistant's help
- Easy to clean
- Can be hung up

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Størrelse / Dimensions / Größe:  
B/W 31 mm

**MC 002-1282** [ HMI: ]  
Master Wall Mount for/for/für Master Board

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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### WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care**'s products with lowered safety rails

Never leave the user with master **care**'s products attached to the lift or with lowered safety rails

### IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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