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I tvivl?: +45 8693 8585

## Washing- and tumble instructions

Washing temperature: **40 - 85°C**  
Tumble dry: **max 60°C**  
Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds

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UK

### WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

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### IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

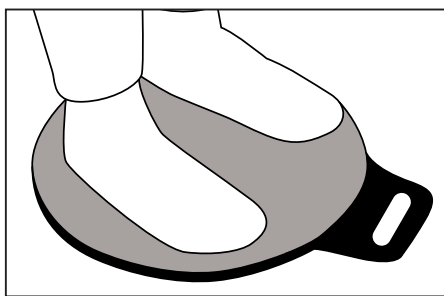
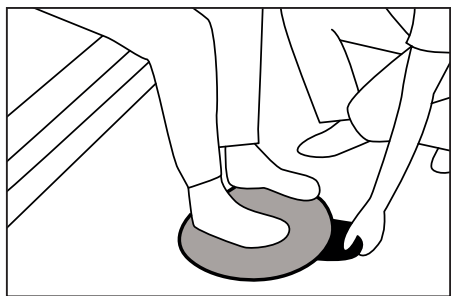
- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the companys at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the toppiece

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


**UK** The user sits and the Master Turn 360/Rotor Master is placed under the user's feet, so that the label faces upward

There must be full contact between the user's feet and the Master Turn 360/Rotor Master

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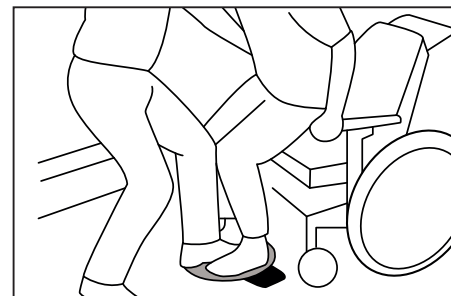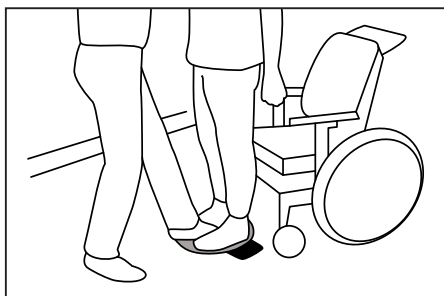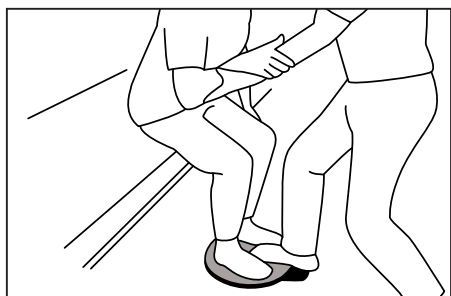**UK** The user gets up with the support of e.g. a helper

The helper rotates the user




The user can sit down aided by the helper

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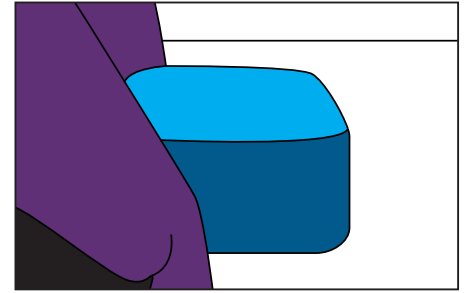
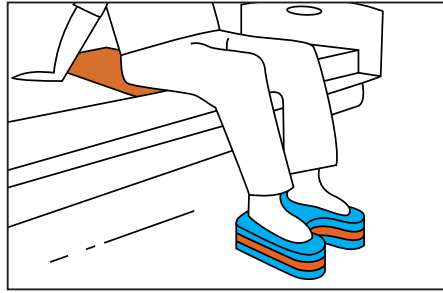
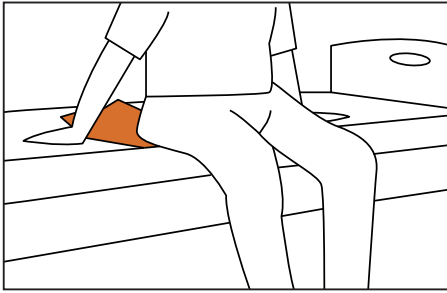
**UK** The user sits on the edge of the bed, to be put into the bed. A Nylon Master is placed under the user's buttocks

The bed is lowered so that there the user's feet make contact with the Foot Master/Step ON. The weight is laid back on the seat nodules, and the user can rotate into the bed

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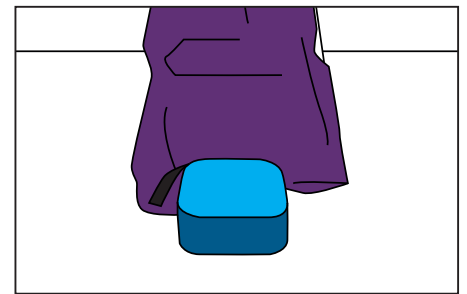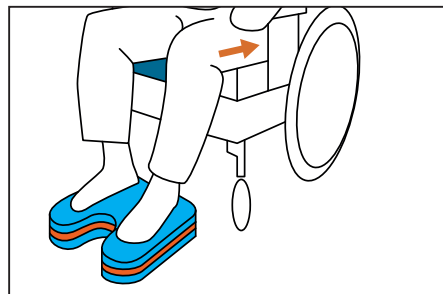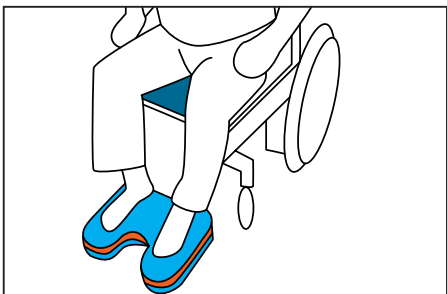**UK** The user sits on the One Way MASTER in the chair and must move further back in the seat. The user's legs are placed on the Foot Master/Step ON, and the weight is laid back on the seat nodules

The user can now use his/her own weight, or with the helper's aid, to push further back in the chair

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# Fordele / Advantages / Vorteile

## Master Turn 360/Rotor Master/Step ON & Foot Master

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UK

- Easy to position under the user's feet
- Enables mobilisation of the user

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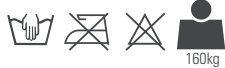
Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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Never leave the user lying on master **care**'s products with lowered safety rails

Never leave the user with master **care**'s products attached to the lift or with lowered safety rails

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- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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