

MULTILIGHT Pro

Instructions for use
of MULTILIGHT Pro lamps

Manufacturer and distributor:

A. SCHWEIZER GmbH
Hans-Böckler-Str. 7
91301 Forchheim
Germany

Phone: +49-9191-72100
Fax: +49-9191-721072
www.schweizer-optik.de

Contents

General information, power supply, cleaning and care	Page 4
Safety and maintenance, battery safety instructions	5
MULTILIGHT Pro – Lamp models	6
MULTILIGHT Pro – Table lamp versions	7
Using MULTILIGHT Pro	8
LED indicators	10
Troubleshooting	10
Technical data	11

Copyright © January 2020
A. SCHWEIZER GmbH
All rights reserved.



Congratulations!

Thank you for choosing this innovative lamp from SCHWEIZER. Made in Germany, this quality product has been manufactured according to currently valid standards and regulations.

Please read these instructions carefully before using the lamp for the first time and observe our safety, cleaning, and care information.

General information

The MULTILIGHT Pro lamp range offers 7 models to choose from: 5 table lamps and 2 floor lamps. All lamps are available in a choice of 3 light temperatures – warm white, neutral white, or cool white. Only the 3in1 models feature all three light temperatures.

Power supply

All MULTILIGHT Pro lamps require mains power. The Akku models use a built-in lithium-ion battery that is recharged via the power adapter. The LED light-emitting diodes are subject to the most stringent selection criteria. They have a lifespan of up to 50,000 hours.

Cleaning and care

Ensure the lamp has been disconnected from the mains supply. Clean the plastic surface with a damp (water or mild detergent) soft and clean cloth. Do not use alcohol, solvents, abrasives or aerosol sprays.

Safety and maintenance

- Do not attempt to repair the MULTILIGHT Pro lamp yourself or to open the housing. This product contains no user serviceable parts. Any unauthorised attempt to service or replace internal components will render the warranty null and void.
- Do not remove the built-in battery. It cannot be replaced by the user.
- Keep your MULTILIGHT Pro lamp away from wet areas, rain and other sources of moisture. Do not rinse your MULTILIGHT Pro lamp with water.
- Charge/operate and store your MULTILIGHT Pro lamp at an ambient temperature of 10-35 °C.
- Only use the supplied charger/power adapter to charge the lamp. The round barrel connector of the charger/power adapter fits easily into the socket of your MULTILIGHT Pro lamp.

Battery safety instructions

We place our batteries on the market in line with legal requirements. Please read the points below on the safe disposal of our batteries:

- The built-in rechargeable battery inside the lamp cannot be removed.
- To prevent harmful effects to the environment and health, it is therefore necessary to dispose of the entire lamp at recycling centres or return it to the point of sale. Returning batteries is required by law and free of charge for you.
- Valuable raw materials such as zinc, iron, nickel and so on can be recovered by recycling used rechargeable batteries.
- The “crossed out wheellie bin” symbol on products indicates that it is illegal to dispose them along with normal refuse.

How to properly charge your battery:

- Charge the battery inside the lamp immediately after it is drained (approx. 3-hour charging time).
- Avoid deep discharging of the rechargeable battery as it could get damaged otherwise.
- Interim charging does not affect the battery service life.

MULTILIGHT Pro – Lamp models



Table lamp
– mains and battery operated
– with 1 or 3 light temperatures



Floor lamp
– mains operated
– with 1 or 3 light temperatures



Table lamp
with extended arm
– mains operated
– in 1 light temperature of choice

Table lamp with extended arm and
clamp
– mains operated
– in 1 light temperature of choice

MULTILIGHT Pro – Table lamp available in 3 versions



Table lamp
with rechargeable battery
in 1 individual light
temperature of choice


Table lamp
mains operated
in 1 individual light
temperature of choice

Table lamp
with rechargeable
battery and 3 light
temperatures


Using MULTILIGHT Pro




Charging/power supply

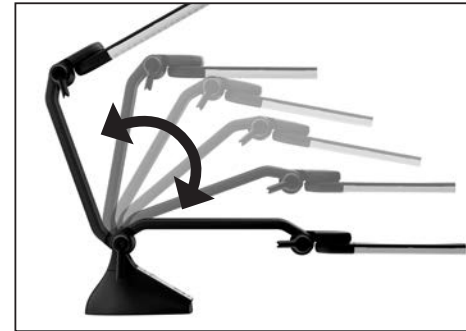


Flexible positioning:
Lamp arm and head can be pivoted and swivelled to multiple positions for optimal lighting

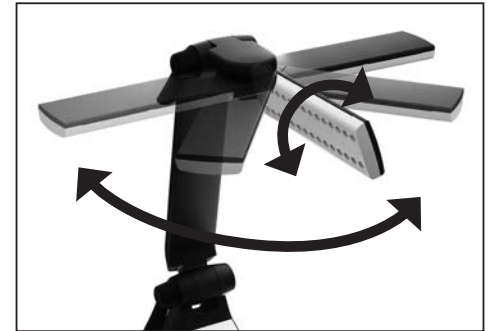

Large ON/OFF button with
with audible feedback

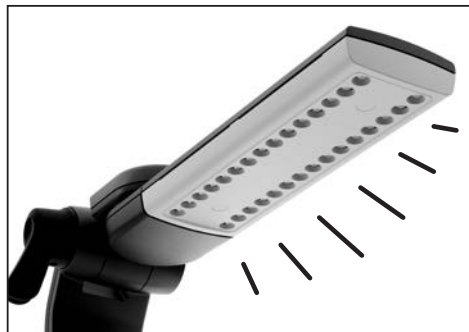

2 brightness levels:
1st click: ON, 100% brightness
2nd click: ON, 70% brightness
3rd click: OFF

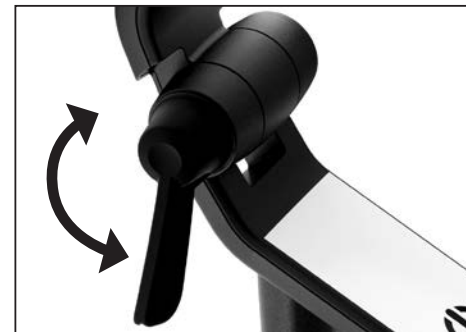

Ergonomic levers facilitate locking and releasing the lamp arm



Only for MULTILIGHT Pro with 3 integrated light temperatures:
Select light temperature (yellow: warm white, white: neutral white, blue: cool white)


D LED indicators

Indicator status description (only for rechargeable battery-operated models)

1. Flashing green light → Battery is charging
2. Solid green light → Battery is fully charged
3. Flashing red light and light turns off every 60 seconds → Battery is empty, charging required



Indicator light flashes/glows solid

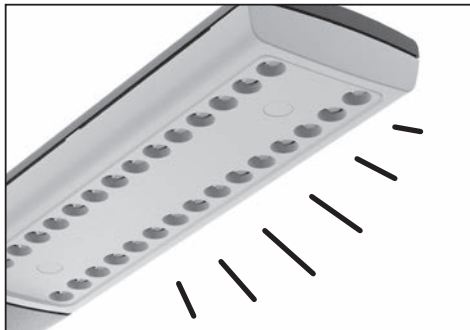

Light goes off temporarily (every 60 seconds)

Troubleshooting

Problem	Solution
1. MULTILIGHT Pro can't be switched on.	Connect the lamp to the power supply before using it. Plug in barrel connector into lamp foot. Then plug power adapter into wall outlet → p. 8.
2. Light goes off temporarily (every 60 seconds).	Rechargeable battery is almost empty. Charge battery → p. 10
3. Lamp switches off during use and LED indicator on control panel flashes red quickly.	Temperature warning active. Switch off and disconnect the device from the mains supply. Leave it to cool, then switch on as usual → pp. 8/10
4. Lamp arm position cannot be changed.	Lift the levers to unlock the joints before positioning the lamp arm. Push levers down to lock the lamp arm in place → p. 9.

Technical data

MULTILIGHT Pro Table mains operated

- Power supply: 100-240 V
- Power adapter 12 V – 2000 mA



MULTILIGHT Pro Table Akku Rechargeable battery

MULTILIGHT Pro Table Akku 3in1 Rechargeable battery, 3 light temperatures

- Lithium-ion battery: 7.4 V – 2600 mA
Operating time: approx. 4 hours
Charging time: approx. 3 hours
- Power supply: 100-240 V
- Power adapter 12 V – 2000 mA



MULTILIGHT Pro Table XL Mains operated, extended arm

MULTILIGHT Pro Table XL Clamp Mains operated, extended arm, clamp

- Power supply: 100-240 V
- Power adapter 12 V – 2000 mA



MULTILIGHT Pro Floor 3in1 Mains operated, 3 light temperatures

MULTILIGHT Pro Floor Mains operated

- Power supply: 100-240 V
- Power adapter 12 V – 2000 mA



