

SAVE THESE INSTRUCTIONS

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

⚠ WARNING

- **DO NOT** install or use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel, before attempting to install this equipment - otherwise, injury or damage may occur.
- All four leg tips **MUST** be in contact with shower/tub floor at all times and on an even surface.
- **ALWAYS** inspect the shower chair to ensure that it is properly positioned and stable before using. **DO NOT** use the shower chair if it is wobbly or unstable.
- Users with limited physical capabilities should be supervised or assisted when using the shower chair.
- The shower chair is **NOT** to be used as a transfer bench, transfer device or climbing device.
- These shower chairs (models H291 and H296) have a weight limitation of 143 kg (315 lb) each.
- Exercise caution when assembling the shower chair to avoid pinching.
- For shower chair model H296 - Ensure that the snap buttons fully protrude through the holes in the back support brackets.
- This product should be used in a surrounding temperature between 10° and 40° C.

⚠ ACCESSORIES WARNING

- Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

⚠ INSTALLATION WARNING

- After **ANY** adjustments, repair or service and **BEFORE** use, make sure that all parts are properly installed and secure.

General

Symbols



Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.



Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.



This product complies with Directive 93/42/EEC concerning medical devices.

Life

The expected life of the product listed in this manual when used in accordance with the safety instructions, maintenance intervals and care instructions is five years. Maintenance and inspection should be done at least every six months.

Intended Use

⚠ WARNING

Risk of injury. Avoid excess impact to the shower chair by dropping the user weight onto the shower chair.

This product is designed solely as an aid for taking a shower and for use in the bathroom. Any other use is prohibited.

Cleaning

NOTE: Before using any cleaning products, test an area on the underside of the seat to make sure the product does not cause discoloration or staining.

1. Remove the legs and backrest from the shower chair.
2. Use water and a mild, non-abrasive cleaner to clean the shower chair.

After Use

Disposal

When the product can no longer be used, the product, accessories and the packaging materials should be sorted correctly.

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

Reuse

⚠ WARNING

Risk of injury. Always have a dealer inspect the product for damage before transferring the product to a different user. If any damage is found, DO NOT use the product. Otherwise injury may occur.




Assembling the Shower Chair

Installing/Adjusting the Leg Extensions

WARNING

Ensure that the shower chair is level and stable before use.

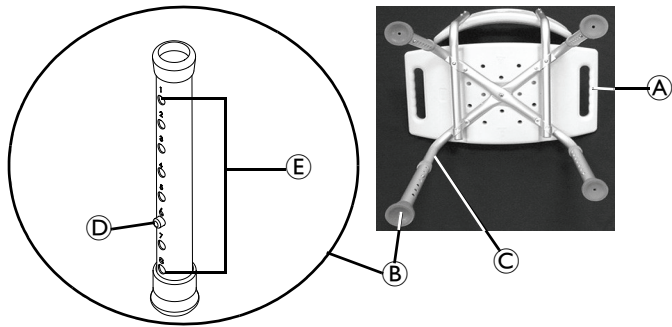


1. Allow the shower chair adjust to room temperature, if necessary.
2. Place the seat **A** upside down on a flat, stable surface (floor or table).
3. Install the leg extension **B** onto the leg **C**.
4. Press the snap button **D** and slide the leg extension up or down to the desired adjustment hole **E**.
5. Ensure the snap button protrudes fully through the adjustment holes of the leg extension.

NOTE: There will be an audible click when the snap button fully engages.

6. Repeat STEPS 2-5 for the remaining three legs.

NOTE: Make sure that all four legs are adjusted to the same height.

Installing the Backrest - Model H296 Only

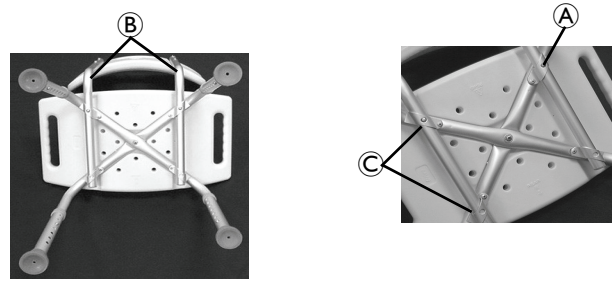

1. Put the shower chair on its side on a flat, stable surface.
2. Press the snap buttons **A** on the back support tubes **B** and slide the back support tubes into the back support brackets **C**.
3. Ensure both snap buttons are fully engaged inside the holes in the back support brackets.

NOTE: There will be two audible clicks when the snap buttons fully engage.

Using the Shower Chair

WARNING

Risk of injury. DO NOT lift the shower chair while the user is seated. The backrest can get loose and injury or damage may occur.