1. Product Name

Kitty Cuddle

2. Product Code

29144

3. Colour

Brown and black

4. Brief Description

Hours of therapy and snuggle time with this weighted neck wrap.

5. Contents

- 1 x cat neck wrap

6. Snoezelen® Stimulations

- Sight
- Touch

7. Best Use

Warm with care or use as a cold compress. Always use under supervision. This is a weighted neck wrap and not a toy. It should always be handled with care. Keep the product as dry as possible.

8. Compatible ROMPA® Products

Not applicable

9. Starting Up

- Unwrap.
- Warm/cool the wrap with appropriate care.

10. Detailed Description

Hours of therapy and snuggle time with this weighted neck wrap. Made from soft fabric. Can be heated in the microwave for use as a heat pack or cooled in the freezer for use as a cold compress. Suitable for supervised use by all ages. This product is not a toy.

11. Safety

Always supervise the use of this product.

- Handle the product with care.
- Do not wring, twist, distort, throw, swing or handle this product roughly.
- This product is not a toy.
- People suffering from nerve damage, circulation problems or paralysis should consult their doctor if in any doubt about the suitability of this product.

12. Technical Specification

Size: 53cmL x 10cmW
Weight: 1kg (2.2 lbs)

13. Installation

1. Unwrap. Either keep the packaging in a safe place out of reach of children and use as required for storage, or dispose of the packaging responsibly. If you do discard the packaging, only do so once you are sure you are keeping the product.
2. If using as a heated wrap, heat the wrap in the microwave for approximately 90 seconds, adjusting this time according to the guidelines supplied with your microwave. If heating the wrap for the first time, heat it gradually.
Please note: Due to the difference in heating times of various microwaves, caution and common sense should be used when heating the wrap. Make sure a safe amount of heat is achieved. Make sure the wrap is a safe and comfortable temperature before using it and before allowing anyone else to use it. Always make

sure the user finds the temperature comfortable. The wrap may be used inside a folded towel for extra safety.

3. Alternatively, the Kitty Cuddle can be used as a cold compress. Place the Kitty Cuddle inside a plastic bag inside the freezer for 1+ hours. As with heating the Kitty Cuddle, make sure the temperature is comfortable before allowing the client to come into contact with it.
4. The Kitty Cuddle can also be used at room temperature – without heating or cooling it.

14. Care and Maintenance

1. Wipe clean with a damp cloth and a mild cleaning product or disinfectant. Allow the wrap to dry thoroughly before re-use.
2. Do not machine wash.
3. Do not immerse in water.
4. Use and store in a dry environment.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

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