

### Product Description & Intended Purpose

This lightweight aluminium Transfer Disc enables safe and comfortable seat to seat transfers. Intended for use by those with reduced mobility and flexibility who require support and assistance when moving. It has been designed to allow both the carer and patient to maintain a natural, ergonomic position during transfers.

### Product Features

- Padded abdominal bar for comfort.
- Height adjustable handle and knee pads.
- Non-slip mat on base platform.
- Raised footpad and lockable brake system to ensure stability.
- Smooth turning mechanism independent of client weight.



### Technical Specification

#### Dimensions (mm)

- |                                   |  |
|-----------------------------------|--|
| • Footplate Diameter: 420         | • Knee Pad Size: 200 x 150                 |
| • Overall Handle Width: 510       | • Knee Pad Height (from centre): 265 — 490 |
| • Lower Handle Height: 750 — 1000 | <b>Product Weight: 22lbs (10kg)</b>        |
| • Upper Handle Height: 890 — 1140 | <b>Maximum User Weight 30st (190kg)</b>    |

### Installation Information: *(Note: for detailed information refer to User Instructions)*

- Please see user instructions regards assembly.

#### **To adjust height of handles/knee pads:**

- Use the hand wheel to loosen the handle shaft. Slide to desired position and secure using the hand wheel and locating holes.

#### **To adjust the height of the knee pads:**

- Use the hand wheel at the back of the knee pad bracket to loosen. Slide to desired position and secure by tightening the hand wheel.

## PTDS037

### Saturn 2 - Transfer Disc

#### Safety Precautions

- The patient must be able to support their own weight and maintain balance, grasp the handles and understand basic instructions.
- Do not use the Transfer Aid as a means of transporting clients.
- Do not leave the client unattended when using the product.
- The carer must act as a counterbalance at all times when the aid is in use by a client.
- Do not exceed the maximum user weight of 30st (190kg). *Please note: whilst the turning aid has been tested to a maximum of 30st (190kg), the maximum client weight is dependent on the relative size and strength of the carer acting as a counter balance.*

#### Cleaning

- Clean regularly, using a general purpose, neutral detergent and warm water.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.
- Ensure the product is fully dry prior to use/storage.

#### Care & Maintenance

- Inspect the transfer aid regularly, pay particular attention to the condition of the swivel plate.
- Check the condition of the knee pads and handle, ensure the hand wheels are correctly located and secure.
- Periodically check the product for signs of wear. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of action.

#### Accessories / Related Products

None

#### FAQ's

None

#### Product Supplied By:

**NRS Healthcare** Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK