



1. Product Name

Activity Board for Wall Mounting

2. Product Code

14350

3. Colour

Multi-coloured

4. Brief Description

An activity board that incorporates a lot of different items.

5. Contents

1 x board
1 x screw packs (taped to underside of board)

6. Snoezelen® Stimulations

- Touch
- Sight
- Sound

7. Best Use

Mount on the wall securely to enable clients to interact with the different activities.

8. Compatible ROMPA® Products

Not applicable

9. Starting Up

- Take product out of box.
- Check item has not been damaged in transit in any way before use (call ROMPA if there are any problems)
- Decide where item is to be positioned on the wall (consider height in relation to users, e.g. access for seated wheelchair users, or standing).

10. Detailed Description

This item has ten different tactile, moveable and visual activities which facilitate use of a range of fine finger movements, grips and activities. As the different elements are utilised, the client will experience different audible and visual input.

11. Safety

- This item should be used under supervision
- Be cautious with moving parts
- For indoor use only, in a dry place

12. Technical Specification

- Size: 114 x 71 x 7cm deep

13. Installation

- Decide the location and height of the board.
- Ensure that anyone interacting with the board has enough space to move and enjoy the activities.
- Ensure that use of the board will not cause an obstruction or hazard (e.g. blocking doorways or corridors)
- Hold the item level and make markings to ensure that the item is installed on the level.
- The board then needs to be fixed securely to the wall using appropriate fixings for the weight of the board and the surface it is to be secured to. Pre-drilled holes are provided for ease.

14. Care and Maintenance

- Use only under supervision.
- Inspect board regularly for wear and tear.
- Check board for damage
- Wipe with a damp cloth and a mild soap/disinfectant solution. Dry before use.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com